


REVISIONS - 601-0141\_\_ (cc)


REV	DESCRIPTION	DATE/BY	ECN NO.

# FOR REFERENCE ONLY


DIMENSIONS FOR MATING SHAFT DETAIL PER DES-0600-M

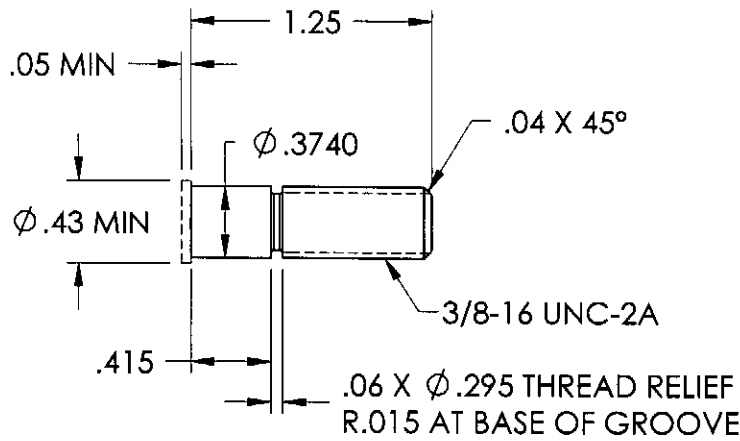

WHEN DESIGNING SHAFT, ALLOW FOR .015 COMPRESSION BETWEEN STEP IN SHAFT AND HANDLE

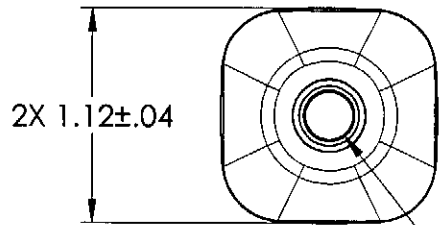


3/8-16 UNC-2B

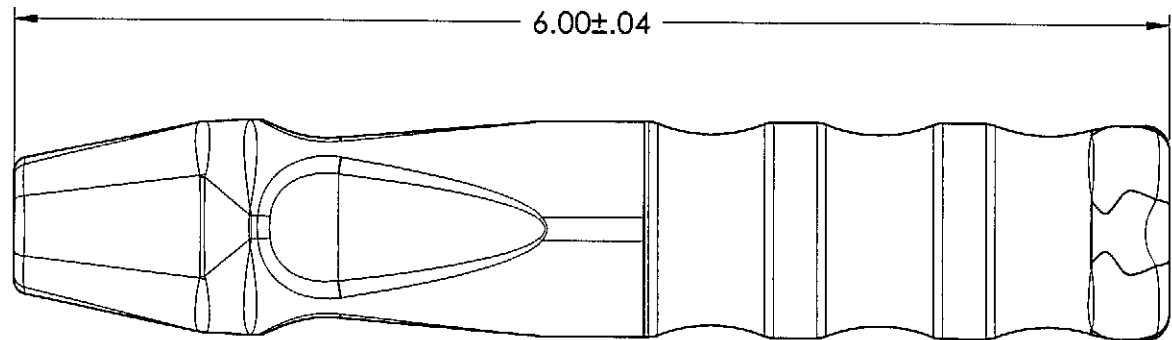

OR DENOTES INSPECTION FEATURE



THIRD ANGLE VIEW

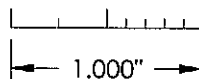

THIS DOCUMENT INCLUDING ALL DATA, SPECIFICATIONS, AND INFORMATION CONTAINED WITHIN IS CONFIDENTIAL AND PROPRIETARY TO BEERE PRECISION MEDICAL INSTRUMENTS BY RETENTION, THE RECIPIENT AGREES NOT TO REPRODUCE, DISCLOSE, OR MAKE USE OF ITS CONTENTS IN ANY WAY WITHOUT PRIOR WRITTEN PERMISSION OF AN OFFICER OF BEERE PRECISION MEDICAL INSTRUMENTS.

MATERIAL 6061-T6 ALUM, SILICONE SST PLUG		UNSPECIFIED TOLERANCES X +/- .020 ANGLE +/- .1° XX +/- .010 MACHINE XXX +/- .005 XXXX +/- .0005 FINISH				5307 95TH AVE KENOSHA, WI 53144 (262) 657-2800	
HEAT TREAT		CONCENTRICITY .005 T.I.R. SQUARENESS .001 PER. IN.		TITLE HDL ASY, CNTR 3-SCLP PLUG F3/8-16		DRAWING NO. 601-0141 (cc)	
FINISH		DO NOT SCALE DRAWING		SIZE A		REV. A	
REMOVEMILL BURRS	BREAK SHARP CORNERS	DRAWN CER	DATE 08/04/04	SCALE 1:1	ILLUSTRATION SHEET 1 OF 1		
PART TO BE FREE OF DIRT, OIL, AND CONTAMINANTS		APPROVED	DATE				